Lansinoh®

FR | Bouts De Seins Protege Mamelons

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INDICATIONS

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Use a nipple shield for its intended use as described in these instructions.

Inspect before each use. Do not use at first signs of damage or weakness.

Never leave packaging and/or components unattended.

If problems or pain occurs, consult your healthcare professional.

Keep out of reach of children when not in use.

Using a nipple shield may encourage your baby to learn to latch whilst waiting to see a tongue-tie specialist and if your baby was premature or sick and has been fed with a bottle, then a nipple shield may help your baby to latch onto your breast.

WARNING

A nipple shield should not be used in the first week after birth unless specifically advised by a breastfeeding expert or a healthcare professional, as its use may affect milk supply.

Before first use, sanitise the nipple shields by boiling in water for 10 minutes.

Before and after each use, wash the nipple shields in hot, soapy water.

Regularly wash the protective case in hot, soapy water and dry thoroughly before storing the nipple shields.

Store in the protective case with the nipple shields facing up.

DIRECTIONS FOR USE

Wash your hands and the nipple shield in clean, hot water and rinse thoroughly before applying to the breast.

For best results, soak the nipple shield in clean, hot water for 1-2 minutes prior to breastfeeding. This makes them more pliable for a better fit on the breast. Drain the water & wait to cool slightly before applying.

Apply a small amount of Lansinoh® HPA® Lanolin only to the rim of the nipple shield (not the entire shield). This will help keep the shield in place leaving both hands free to position and attach your baby.

Partially turn the nipple shield inside out by bringing the long edges toward the shield's nipple.

The cut-out area of the nipple shield should be placed where your baby's nose will be when feeding. This promotes skin-to-skin contact between you and your baby. Your baby's nose should not be touching your breast or the nipple shield.

Bring the shield to your breast, lining your nipple up with the shield's nipple opening.

Push the rim of the shield toward your breast timing it right side out and enclosing as much of your nipple as possible in the nipple opening.

Storing conditions: Product should be stored in dry place away from direct sunlight.

Disposal method: This product is a medical device and should be discarded as a medical product according to the local procedures and regulations.

Serious events experienced from using this product should be reported immediately to Lansinoh Laboratories and to the local competent authority.

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BEWAHREN SIE DIE STILLHÜTCHEN MIT DER HÜTCHENSPIZITZE NACH OBEN ZEIGEND IN DER TRANSPORTBOX AB.