Lansinoh®

LANSINOH® BREASTMILK STORAGE BAGS

Consumer package insert

Important! Please read the following instructions in their entirety before using the storage bags

It takes a tremendous commitment to breastfeed and an enormous effort to provide breastmilk in your absence. You are doing everything you can to make sure your baby gets the very best and we want to make sure that happens as well. In addition to the directions below, it is important that you handle your stored breastmilk with all the care and attention you have given to producing it in the first place.

Lansinoh breastmilk storage bags are made of the highest quality. While it is impossible to guarantee 100% that no leakage will occur, this package insert includes instructions and guidelines for safer and optimal use of the storage bags, including the filling, storage and defrosting of the milk storage bags and the milk.

IMPORTANT: We have chosen virgin plastic, which contains no chemical plasticizers, for the production of our storage bags to ensure that no harmful elements will ever migrate into your breastmilk. This material is rigid in shape because there are no plasticizers to help it stretch as you might observe in other plastic bags. It is important to realize that when your milk is frozen it may stretch the bags beyond their initial size. This means that the bags could be more susceptible to tearing if they are not handled properly. For instance, if a bag of milk is dropped or strikes the edge of the counter or side of the freezer, it could weaken the plastic at the point of impact and cause it to leak upon thawing. With this in mind, it is a good idea to handle the frozen milk with care while in the freezer and when it is removed from the freezer and prepared for your baby.

Please note: Each bag is intended for single use. Lansinoh breastmilk storage bags are pre-sterilized, once the safety seal at the top of the bag is opened; the storage bag is no longer sterile.

Storage Bag Instructions

- Tear off the safety seal at top of the bag and open by gently pulling the zipper seal apart.
- Carefully pour your milk into the storage bag. Lansinoh breastmilk storage bags are pre-sterilized and do not need to be washed before use.
- Before closing the bag, read the fill quantity by means of the imprinted scale, close the bag and mark quantity in the inscription area. Do not fill the bag more than the maximum quantity allowed (180 ml).
- To avoid freezer burn, be sure to compress the air out of the bag before sealing. To do this, take the bag between your forefinger and middle finger (like scissors) and press the air above the liquid level towards the zipper. However it is important to leave a small amount of air in the bag to allow for expansion during freezing process.
- Seal the bag by compressing the double seal. Repeat the procedure to ensure safe sealing.
- When pouring the milk in to the bags and after closing the bag and thoroughly sealing, make sure there is no leakage of milk from the bag.
- Write the date the milk was expressed on each bag, on the outer write-on tab. Use older milk first.
- Do not store breastmilk in freezer or refrigerator door. The door tends to be the warmest part of the refrigerator/freezer.
- Each bag is intended for single use only.

Storage of breastmilk

Check with your midwife, health visitor or breastfeeding counselor for specific storage instructions. When you are freezing your breastmilk, do not fill containers more than 3/4 full to allow for expansion from freezing (as provided above for Storage Bags). Date containers with the date you expressed your milk for further reference.

Importer: Altman Health General Partnership, 2 Moshe Aviv, Or Yehuda, Israel.

Manufacturer: Lansinoh laboratories.

Country of origin: Thailand.

Costumer Service: *6505, altman.co.il

Recommended breastmilk storage times (According to the Israeli Ministry of Health recommendations, June 2011)*:

Storage	Temperature	Recommended storage time	Possible storage time	Comments
Room temperature	16 – 19 °C	3 to 4 hours	Up to 6 hours	-
Sealed Cooler with an ice pack	-	-	Up to 24 hours	Keep continuous contact with the cooling pad. Move to fridge as soon as possible
Fresh milk in the refrigerator	4°C or less	3 days	5 to 8 days	Store at the back part of the fridge
Frozen and thawed milk in the refrigerator	4°C or less	-	Up to 24 hours	Store at the back part of the fridge
In the freezer with a separate door or a stand - alone freezer (deep freeze)	4°C or less	6 months	Up to 12 months	Store at the back part of the freezer

*Based on: ABM Clinical Protocol No. 8, March 2010

Recommended breastmilk storage times for thawed breastmilk:

Storage	Temperature	Recommended storage time
Refrigerator – Sealed breastmilk storage bag that hasn't been opened	4°C or less	Up to 24 hours
Refrigerator – Sealed breastmilk storage bag that has been opened	4°C or less	Up to 12 hours

It is important to handle the storage bags with extra special caution while in the freezer and when taking out and defrosting.

Instructions and recommendations for safer storage of milk bags:

Storage Instructions

- DO NOT refreeze thawed breastmilk.
- DO NOT store breastmilk in freezer or refrigerator door. The door tends to be the warmest part of the refrigerator/freezer.
- Date all breastmilk and use the oldest breastmilk first.
- Freeze the bags on their side (and not up right), as this allows the milk to become more homogenous and for faster thawing.
- Store the bags in the freezer in a separate utensil with a smooth surface, rather than directly on the surface of the freezer, thus protecting the bags from friction with other products in the freezer.
- It is NOT recommended to store the bags in a tight and narrow space, nor very close or next to other storage bags.

Instructions for defrosting

1. To preserve the precious substances contained in your breastmilk, defrosting should be carried out as gently as possible – either very slowly over 24 hours in the fridge, or at room temperature.
2. Do not use hot or boiling water or a microwave to defrost frozen breastmilk! If necessary, you can defrost your breastmilk under running cold or lukewarm tap water or in a bottle heater (max 37°C).
3. It is recommended to defrost the bags whilst on their side.
4. It is not recommended to defrost the bags by placing in a bowl of water.
5. Before opening the defrosted bag, make sure there is no milk leakage.
6. After defrosting, you can put the milk back into the fridge until consumption. Unopened, the bag containing defrosted breastmilk can be kept in the fridge and consumed within 24 hours. A sealed breastmilk storage bag that has been opened can be kept in the fridge and should be consumed within 12 hours.
7. The defrosted milk might separate into different layers. This is a normal process as breastmilk is not homogenous.
8. After defrosting the milk, pour into a feeding bottle.
9. After feeding the baby with the defrosted breastmilk, the remaining milk must be discarded.

IMPORTANT!

- Before feeding baby with thawed breast milk it is always important to check the temperature of the milk and also to check the taste of the milk to make sure it hasn't gone off.
- Each bag is intended for single use.

In any case, while pouring the milk into the bag and during defrosting of the milk in bags after freezing, please take caution and make sure there is no leakage of milk from the bag. In any case that you do notice a leakage, it is recommended to put the bag containing the milk into an extra, new storage bag to ensure that the milk does not spill.

Caution! For the sake of your child's safety and health:

- All parts of the storage bags that are not in use must be kept out of babies and children's reach.
- Throw away used breastmilk storage bags.

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